## FR: Mode d'emploi

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# EN: Operating Instructions

Before using CBC Percuflex, read the operating instructions carefully and keep them for future reference.

Check the product for any visible damage and functionality prior to each use. In the event of defects or malfunctions, discontinue use of the product.

## Intended use:

This medical device should only be used by trained personnel. Reflex hammer with turnable head for reflex testing and percussion.

## Contraindication/side effect:

None known.

## Patient target group:

No restrictions.

## User:

This medical device should only be used by trained personnel.

## Characteristics:

- Combines the individual advantages of a Babinski, Trömner, Buck and Berliner reflex hammer
- Individually adjustable
- Slightly flexible handle
- Pointed handle end for plantar reflex
- Available in two weights: 75 g and 150 g
- Total length: 21 cm
- Adjustable pivoting head rotates 360° for different striking positions

## Cleaning:

Do not use automated cleaning methods. Wipe CBC Percuflex with a lint-free cloth dampened with a pH-neutral detergent.

## Disinfection:

Do not use automated disinfection methods. To prevent cross-contamination between patients, wipe disinfection of the reflex hammer is recommended after every use. Use disinfectants approved by the German Society of Hygiene and Microbiology DGHM, avoiding disinfectants based on alcohol.

## Disposal:

Please dispose of CBC Percuflex in accordance with your local waste disposal regulations.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the national competent authority.

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