REF 2260
kurz / short 18 cm

REF 2261
lang / long 33 cm

CBC Hebammenstethoskop CBC Pinard Stethoscope

Kimetec GmbH
Gerlinger Straße 36-38
71254 Ditzingen
Tel.: +49 (0) 71 56 / 1 76 02 - 200
Fax: +49 (0) 71 56 / 1 76 02 - 500

www.kimetec.de
info@kimetec.de

GB2050002M2021-01

Kimetec

CBC Hebammenstethoskop

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CBC Pinard Stethoscope

Operating instructions

Read these instructions carefully before using the pinard stethoscope and keep them in a safe place for future reference.

Intended use:

The Pinard stethoscope is a diagnostics means to assess acoustic phenomena in the body, in particular fetal and maternal heart sounds.

Contraindication/side effect:

None known.

Patient target group:

Pregnant women.

Users:

This medical device should only be used by trained personnel.

Application:

Check the product for any visible damage and functionality prior to each use. In the event of defects or malfunctions, discontinue use of the product.

Place the flat side of the pinard on your ear.

Material:

The stethoscope for midwives is made from natural, untreated beechwood. Each Pinard is a one-off, being unique in grain and colouring. The hand-turned stethoscope is coated with solventfree oil-wax.

Cleaning:

Wipe off the pinard stethoscope with a lint-free cloth damped with a pH neutral detergent. Product is not suited for automated cleaning procedures.

Disinfecting:

In general, it will be sufficient to disinfect the surfaces which come into contact with patients with a disinfecting wipe after each use. Use disinfectants approved by the German Society of Hygiene and Microbiology DGHM, avoiding disinfectants based on alcohol. Do not use automated disinfection methods.

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Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the national competent authority.