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Available colours

150 Gramm 150 gram	75 Gramm 75 gram		
lila purple	REF 920320	lila purple	REF 920420
rot red	REF 920330	rot red	REF 920430
mint mint	REF 920340	mint mint	REF 920440
limette lime	REF 920350	limette lime	REF 920450
blau blue	REF 920360	blau blue	REF 920460
orange orange	REF 920380	orange orange	REF 920480
gelb/lila yellow/purple	REF 920370	gelb/lila yellow/purple	REF 920470
limette/blau	REF 920390	limette/blau	REF 920490
lime/blue		lime/blue	

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Operating Instructions

Before using CBC Percuflex, read the operating instructions carefully and keep them for future reference.

Check the product for any visible damage and functionality prior to each use. In the event of defects or malfunctions, discontinue use of the product.

Intended use:

This medical device should only be used by trained personnel. Reflex hammer with turnable head for reflex testing and percussion.

Contraindication/side effect:

None known.

Patient target group:

No restrictions.

User:

This medical device should only be used by trained personnel.

Characteristics:

- Combines the individual advantages of a Babinski, Trömner, Buck and Berliner reflex hammer
- Individually adjustable
- Slightly flexible handle
- Pointed handle end for plantar reflex
- Available in two weights: 75 g and 150 g
- Total length: 21 cm
- Adjustable pivoting head rotates 360° for different striking positions

Cleaning:

Do not use automated cleaning methods. Wipe CBC Percuflex with a lint-free cloth damped with a pH-neutral detergent.

Disinfection:

Do not use automated disinfection methods. To prevent cross-contamination between patients, wipe disinfection of the reflex hammer is recommended after every use. Use disinfectants approved by the German Society of Hygiene and Microbiology DGHM, avoiding disinfectants based on alcohol.

Disposal:

Please dispose of CBC Percuflex in accordance with your local waste disposal regulations.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the national competent authority.